

Open the door to simplification



Essentia<sup>®</sup>  
Universal shade

from GC

1 shade, 3 strong options  
for all posterior indications

**GC**


# Essentia<sup>®</sup> Universal shade from GC

## 1 shade, 3 strong options for

Essentia from GC marked a paradigm shift in restorative dentistry, introducing a new concept in aesthetic restorations by using just seven shades, thus opening the door to a world of simpler restorative dentistry.

With the **Essentia Universal shade**, GC throws the door to simplification wide open. Essentia Universal shade is **the ultimate solution for posterior restorations: one shade with an amazing blending effect, now available in three viscosities**. Choose between a paste, an injectable & a very flowable viscosity **depending on what is most useful for each case**. From cervical to occlusal restorations, all versions of the Universal shade are **strong enough for all cavity classes**, without limitations.

Flexural strength & 3-body wear of the three viscosities of Essentia Universal shade (Source: GCC R&D data)

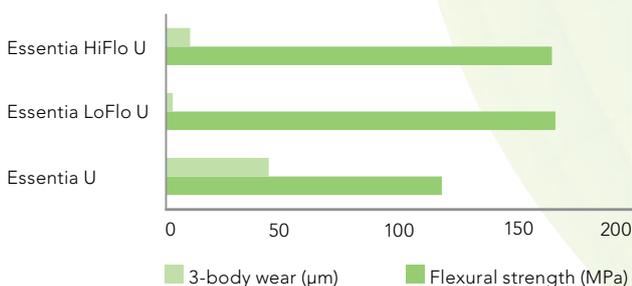

Thanks to a proprietary filler treatment and to the use of ultra-fine particles, the physical properties of Essentia HiFlo & Essentia LoFlo are even higher than those of Essentia Universal, confirming that all products can be safely used for all indications.



Preparation of the small Class I cavities



Restoration using Essentia HiFlo Universal



Post-operative view showing a great blending with natural teeth



# all posterior indications



Air abrasion of the non-carious lesions



Restoration with Essentia LoFlo Universal



Post-operative view after polishing



**Essentia HiFlo** is a fluid composite with a **high wettability**, perfect for easy adaptation to preparations and flow into narrow cavities. It offers great handling properties together with superior strength and a high radiopacity of over 300% facilitating the diagnosis of secondary caries.



**Essentia LoFlo** is an injectable composite with a **low flowability**, optimal when a thixotropic material is needed – for instance for cervical restorations. It promises minimal wear, ensuring long-lasting restorations even on the occlusal surface.



**Essentia** is a paste composite with a simplified 7-shade concept. Its Universal shade displays a great blending effect together with a **packable consistency**: perfect for mono-shade posterior restorations.



A direct approach was selected for this challenging post-endodontic case



Restoration with everX Posterior as a dentin core and Essentia Universal as the outer shell



Six-month recall showing good shade integration with the surrounding dental tissues

# Essentia<sup>®</sup> Universal shade from GC

1 shade, 3 strong options for all posterior restorations



## Essentia HiFlo

Universal light-cured composite  
with high flowability



## Essentia LoFlo

Universal light-cured composite  
with low flowability



## Essentia

Universal light-cured composite  
with a packable consistency

900739 Essentia HiFlo Syringe, U (1 x 2mL)  
900741 Essentia LoFlo Syringe, U (1 x 2mL)  
900970 Essentia Syringe, U (1 x 2mL)  
900998 Essentia Unitips, U (15 x 0.16mL)

## Also discover G-Premio BOND

GC's universal bonding featuring zero compromises

009030 G-Premio BOND Kit 5mL  
009036 G-Premio BOND Bottle Refill 5mL  
009034 G-Premio BOND unit doses,  
50 pieces (0.1mL)



e s s e n t i a

**GC EUROPE N.V.**  
Head Office  
Researchpark  
Haasrode-Leuven 1240  
Interleuvenlaan 33  
B-3001 Leuven  
Tel. +32.16.74.10.00  
Fax. +32.16.40.48.32  
info@gceurope.com  
http://www.gceurope.com

**GC UNITED KINGDOM Ltd.**  
12-15, Coopers Court  
Newport Pagnell  
UK-Bucks. MK16 8JS  
Tel. +44.1908.218.999  
Fax. +44.1908.218.900  
info@uk.gceurope.com  
http://uk.gceurope.com

**GC**