

3M Science.
Applied to Life.™

3M™ RelyX™ Universal
Resin Cement

Goodbye complexity.



The two that can do it all.

3M™ RelyX™ Universal Resin Cement works both as a standalone, self-adhesive cement and as an adhesive cement when combined with **3M™ Scotchbond™ Universal Plus Adhesive**.

So simple yet so powerful.

- Two components for virtually all indirect and direct bonding indications
- Outstanding self-adhesion of RelyX Universal Resin Cement
- Scotchbond Universal Plus Adhesive further enhances bond strength to all substrates
- Unique 3M automix syringe
- Easy excess clean-up
- High esthetics



Together they eliminate the hassle and confusion of multiple resin cements, primers and adhesives, **simplifying your direct and indirect restorative workflows**. With fewer products to stock and replenish, you save space, time and money.

Game-changing syringe.

Proprietary 3M design takes automix cement applications to the next level. Developed with your needs in mind for easy dispensing, cleaning and storage. The small, low-waste 3.4g syringe provides enough material for around 15 applications.

Easy excess clean-up.

Efficient to the last step. Its unique chemistry ensures excess outflow stays put at the restoration margins, and can be easily removed after tack curing.



Self-adhesive cementation of a 3M™ Lava™ Esthetic Fluorescent Full-Contour Zirconia bridge; Image courtesy of Dr. Jan Gueth, Germany

Less stock, more savings

95%

of dentists agreed the universal character of RelyX Universal Resin Cement helps to **reduce stock and save costs**.

Easier for operatory staff

97%

of dentists agreed using one universal versus multiple resin cements is **easier for operatory staff**.

* per application compared to currently available standard automix systems; ** compared to currently used automix syringes; *** Source: Field Evaluation EU/USA conducted by 3M

“Excellent hygienic dispensing system with little waste.”
M.W., Great Britain***

80%

less cement waste* with Micro Mixing Tip


More hygienic.**
Self-sealing for storage without used mixing tip.

Thin, long elongation tip for easy root canal application



Outstanding self-adhesion for maximized efficiency.

With its superior self-adhesive bond strength to dentin, RelyX Universal Resin Cement reliably treats most cases with no need for an additional adhesive or primer. For light- or self-cure.

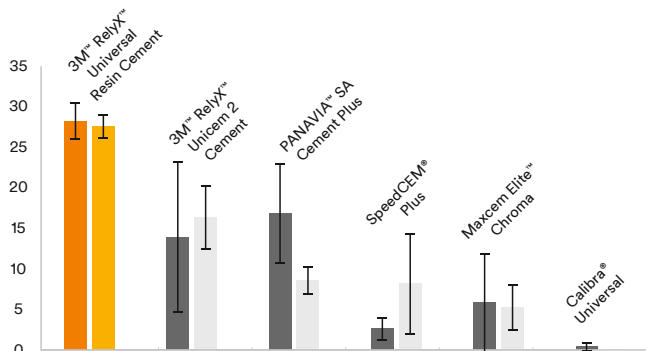

Shear bond strength (MPa) to dentin after artificial aging
R. Afutu, M. Abreu, G. Kugel, Tufts University, J. Dent. Res. Vol 98A, No 3629, 2019


“Easy handling, few components and therefore less potential for error.”
J.L., Germany***

More ergonomic shape with
50% less plastic waste*

The strong partner for adhesive cases.

3M™ Scotchbond™ Universal Plus Adhesive is the first radiopaque, all-in-one universal adhesive for all direct and indirect bonding indications. And, it is a fully aligned system: the adhesive is cured by the cement, so no light-cure is needed.

Scotchbond Universal Plus Adhesive serves as universal primer for all materials including glass ceramics and further boosts the cement's excellent self-adhesive bond e.g. to zirconia.

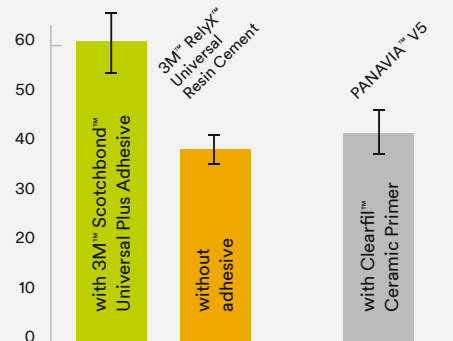


Self-cure shear bond strength (MPa) to zirconia.
C. E. Sabrosa et al., J. Dent. Res. Vol 99A, No 1838, 2020



Virtually no post-operative sensitivity

99% of dentists reported that RelyX Universal Resin Cement shows virtually no post-operative sensitivity***

High esthetics with four color-stable, fluorescent shades

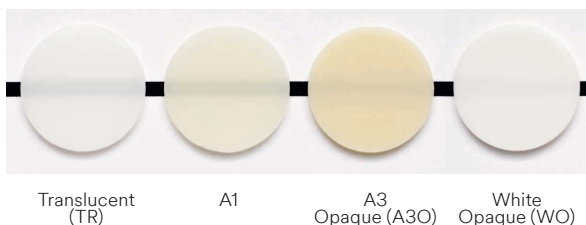

Ordering information

3M™ RelyX™ Universal Resin Cement

Item no. Product information

Intro Kit

56968 4x 3M™ RelyX™ Universal Resin Cement syringes shades TR, A1, A3O, WO (3.4g each), 60x 3M™ RelyX™ Universal Micro Mixing Tips, 20 Elongation Tips, 1 vial 3M™ Scotchbond™ Universal Plus Adhesive (5 ml), 1x 3M™ Scotchbond™ Universal Etchant (3 ml), 25 blue etchant tips, 50 orange microbrushes, step-by-step guides

Trial Kits

Each trial kit contains: 1x 3M™ RelyX™ Universal Resin Cement syringe (3.4g), 15x 3M™ RelyX™ Universal Micro Mixing Tips, 5 Elongation Tips, 1 vial 3M™ Scotchbond™ Universal Plus Adhesive (1.5 ml), 1x 3M™ Scotchbond™ Universal Etchant (3 ml), 25 blue etchant tips, 50 orange microbrushes, step-by-step guides

56969 **Shade TR** **56970** **Shade A1**

Refills

Each refill contains: 1x 3M™ RelyX™ Universal Resin Cement syringe (3.4g), 15x 3M™ RelyX™ Universal Micro Mixing Tips, 5 Elongation Tips

56971 **Shade TR** **56973** **Shade A3O**

56972 **Shade A1** **56974** **Shade WO**

Value Packs

Each value pack contains: 3x 3M™ RelyX™ Universal Resin Cement syringes (3.4g each), 45x 3M™ RelyX™ Universal Micro Mixing Tips, 15 Elongation Tips

56977 **Shade TR** **56978** **Shade A1**

Mixing Tips

56975 30x 3M™ RelyX™ Universal Micro Mixing Tips

56976 15x 3M™ RelyX™ Universal Micro Mixing Tips with Elongation Tips

3M™ Scotchbond™ Universal Plus Adhesive

Vial

41293 **Intro Kit:** 1x 3M™ Scotchbond™ Universal Plus Adhesive vial (5 ml), 50 disposable brushes, 1x 3M™ Scotchbond™ Universal Etchant (3ml), 25 dispensing tips

41294 **Refill:** 3M™ Scotchbond™ Universal Plus Adhesive vial (5 ml)
1x 5 ml vial **41295** **3x 5 ml vial**

Unit Dose

3M™ Scotchbond™ Universal Plus Adhesive Unit Dose, 1x 3M™ Scotchbond™ Universal Etchant (3ml), 25 dispensing tips

41297 **Intro Kit: 50x L-Pop** **41299** **200x L-Pop**

41298 **100x L-Pop** **41304** **400x L-Pop**

3M™ Scotchbond™ Universal Etchant Etching Gel

41263 2x 3M™ Scotchbond™ Universal Etchant (3ml each), 50 dispensing tips



3M™ RelyX™ Universal Resin Cement Intro Kit (56968)



3M™ RelyX™ Universal Resin Cement Trial Kit TR (56969)



3M™ RelyX™ Universal Resin Cement Refill TR (56971)



3M™ Scotchbond™ Universal Plus Adhesive Refill (41294)



3M™ Scotchbond™ Universal Plus Adhesive Unit Dose 100 L-Pop (41298)

3M.com/RelyXUniversal

3M.com/ScotchbondPlus



3M Oral Care • 2510 Conway Avenue • St. Paul, MN 55144-1000 USA • Phone 1-800-634-2249 • 3M.com/dental • 70-2013-7322-5
3M, Lava, RelyX and Scotchbond are trademarks of 3M Company or 3M Deutschland GmbH. Printed in USA © 3M 2020. All rights reserved.